



HOW TO USE TOTIM®

During the moulding and use of TOTIM® cushion, it is essential to follow the indications to obtain the best final foam density and homogeneity.

- 1 Open the box.
Place TOTIM® on a rigid flat surface.
Identify the symbol 'SIDE UP' on the cushion.

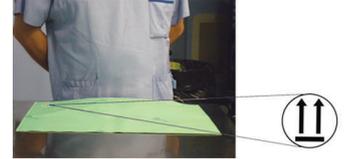

- 2 Identify the inner bag with the two pockets, A and B.
Apply steady vertical pressure with both hands on one pocket in the direction of the other pocket so as to tear the central safety sealing.

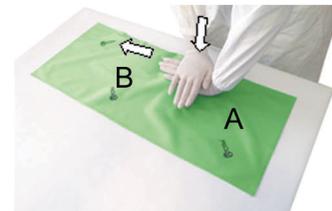

- 3 Energetically mix for about 30 seconds keeping a hand on both pocket (occasionally using one hand vertically to shift the two components from one side to the other) until an increase in the volume is seen. Once inflated, wait until the inner bag opens autonomously.



INCREASE OF VOLUME

LEAKAGE OF FOAM

- 4 With both hands completely remove the foam from the inner bag by pressing toward the inside the cushion.



- 5 Spread the foam on all the cushion surface, then position the patient after 30/40 seconds. The patient's head must be carefully positioned in the middle. Keep the position for about 3 minutes to allow the homogeneous expansion of the foam inside the cushion.



WARNING: STAND STILL IN THE POSITION – DON'T TOUCH THE CUSHION!